Please Read This Book First

ORAQUICK®

HIV SELF-TEST

HIV, Testing & **Me**

Manufactured by:

OraSure Technologies, Inc.

220 East First Street
Bethlehem, PA 18015 USA
www.OraSure.com

© 2025 OraSure Technologies, Inc.

OraQuick®, logo design, and configuration are trademarks of OraSure Technologies, Inc.

Please call our toll free number (1-866-436-6527) regarding any questions about:

- Whether the test is right for you to take
- How to use the test
- How to read your test results
- What to do if you are not sure of your result
- What to do once you have your test results
- How to find local doctors or clinics for additional testing or treatment options

All calls are completely confidential.

Or go to www.oraquick.com for more information on:

- HIV/AIDS
- Product Information
- Product use video
- Testing, counseling and care referrals
- Where to buy

www.oraquick.com

**If you have any questions about this HIV test or your test results,
call toll free 1-866-436-6527 or go to www.oraquick.com**

Things to consider before taking the OraQuick® HIV Self-Test

You probably have some concerns about your HIV risk and what your test result would be – that's totally normal! Learning about your HIV status is a giant step towards **taking control of your health**. No matter what the result, knowing your HIV status will help you to learn how to take better care of yourself. It takes a lot of courage to get to this point – this booklet will help you to better understand HIV, and what to do with the result you get on this test.

First, let's talk about some background information on HIV.

What is HIV?

HIV (Human Immunodeficiency Virus) is a virus that attacks the body's immune system and decreases your body's ability to fight off other sorts of diseases. There are three stages of HIV infection:

Stage 1: The acute phase, where the virus starts to replicate in your body. This typically happens within 2-4 weeks of infection. There is a lot of HIV virus in your body during this time, which increases the risk of transmitting HIV to others.

Stage 2: The chronic phase, where HIV continues to multiply in your body but at a slower rate. You may or may not have any symptoms of HIV during this period. Treatment can reduce the amount of HIV and make you feel healthy, but will not completely eliminate HIV from your body.

Stage 3: AIDS (Acquired Immunodeficiency Syndrome), where the virus has severely damaged your immune system and your body cannot fight off opportunistic infections. AIDS generally occurs with untreated HIV, so regular testing can be helpful to prevent AIDS.

HIV is a chronic condition. Medical treatment can **help you stay healthy**; many people living with HIV experience very few medical complications and, with an undetectable viral load, there is effectively no risk of transmitting HIV to an HIV-negative partner through sex.

Anyone can get HIV – HIV is transmitted through contact with bodily fluids including blood, semen, pre-seminal fluids, rectal fluids, vaginal fluid, and breast milk. HIV can NOT be spread by air or water, saliva, tears, sweat, casual contact like shaking hands, or kissing.

In the United States, HIV is mainly transmitted through:

- Having anal or vaginal sex with someone who has HIV without using a condom or taking medicines to treat HIV
- Sharing injection drug equipment, such as needles, with someone who has HIV

HIV can also spread from someone with HIV to their child during pregnancy, childbirth, and breastfeeding. If you are pregnant, your healthcare provider will perform an HIV screening test during your pregnancy, unless you decline.

Protection & Prevention

Talk about HIV and sexually transmitted infections

(STIs) with your partner! Conversations about sex, sexually transmitted infections, and HIV can be scary, and can also help empower you to take control of your own health. Having open conversations about HIV status, sexual practices, and STI testing can help you and your partner(s) to navigate your sex life in a way that helps you to make informed decisions.

Consider using condoms or other forms of barrier protection. Condoms and dental dams can help to reduce the chance of you contracting HIV or an STI when you're having sex.

Learn about PrEP and consult a clinician on whether it's right for you. PrEP, or pre-exposure prophylaxis, is medicine people at risk for HIV take to prevent them from getting HIV from sex or injection drug use. PrEP can stop HIV from taking hold and spreading throughout your body. PrEP is highly effective at preventing HIV when taken as indicated. There are pill and injectable versions of PrEP. You can talk to a clinician about PrEP if you are interested in exploring it further.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

Learn about U=U. U=U (Undetectable = Untransmittable) means that people with HIV who achieve and maintain an undetectable viral load—the amount of HIV in the blood—by taking antiretroviral therapy (ART) daily as prescribed cannot sexually transmit the virus to others. If you have sexual partners who have HIV, discussing U=U can help you to make informed decisions about the activities you engage in. The OraQuick® HIV Self-Test is not intended for use by people who have already tested positive for HIV, and should not be used to determine whether a person's viral load has been reduced to an undetectable level.

When possible, do not share equipment to inject drugs.

When you share equipment used to inject drugs, you have an increased risk of contracting HIV. There are ways to reduce your risk while injecting drugs, such as using a new syringe every time you inject drugs. Local resources, such as sterile syringe programs, can give you more information about ways to keep yourself safe.

What if your test result is negative?

This test works by detecting your body's response to an HIV infection. If your test result is negative, that means that no response to an HIV infection was picked up by the test. If it has been at least 3 months since your last potential exposure to HIV (through one of the ways it can transmit, listed above), then you are likely HIV negative. It's important to note that any test can sometimes have false results. For this test, 8 out of 96 people who had HIV reported a negative test result, even though they had HIV. It's important to be aware that you sometimes may get a false result. If your test is negative but you suspect that you might be HIV-positive, then you should speak with a healthcare provider.

A negative test result is information that you can use to make decisions about your health. What you do with this information is totally up to you!

What if your test result is positive?

It is very important to understand that if you get a positive result, a healthcare professional must confirm your OraQuick® HIV Self-Test result.

However you're feeling right now is okay. It can be scary to find out that you may have a chronic condition that you will need to manage. That's okay! Today there are highly effective drugs that can suppress HIV and allow you to live a long healthy life. Take the time that you need to process how you are feeling. Reaching out to friends and family, mental health professionals, support groups, or doing activities like journaling are all potential things that you may try as you decide what next steps make sense for you.

You're not alone. There are many people and organizations in communities around the country who are ready, willing and able to help you. The key is to ask for help as soon as possible. If you would like to be linked to HIV-related medical or mental health services near you, please call the OraQuick® Support Center toll-free at 1-866-436-6527.

Knowing is the first step to taking care of yourself.

By knowing that you are HIV positive you can seek treatment, stay healthy longer or improve your current health. It can also prevent you from accidentally passing it on to someone else. While there may be some changes in your life, you will still be able to engage in the activities you enjoy, from sex to exercise to other hobbies, and do the sort of work that you want. With an undetectable viral load, you cannot transmit HIV through sex.^[1] By taking medication, you can monitor your health and stay healthy. If you want to have children, there is less than a 1% chance that you will transmit HIV to your baby, with the appropriate medication and medical supervision.

1. <https://www.nih.gov/news-events/news-releases/science-clear-hiv-undetectable-equals-untransmittable>

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

HIV is a chronic condition, and medical treatments are available to help people live long healthy lives.

Having HIV does not mean that you have or will get AIDS. It is important that you see your doctor or healthcare professional as soon as possible. You should let your doctor or healthcare professional know the result of your OraQuick® HIV Self-Test so they can confirm your results. Your doctor or healthcare professional will run several tests to determine your current state of health and may want you to consider medication to help your body fight HIV. If you need help finding a doctor or healthcare professional, please call the OraQuick® Support Center toll-free at 1-866-436-6527. Your call will be completely confidential.

The good news is that you may be able to get help paying for treatment. The ability to receive payment for HIV treatment may vary depending on where you live. Talk to your doctor or call the OraQuick® Support Center toll-free 1-866-436-6527 for more information.

How to find out more about HIV or HIV prevention?

For more information, please call a representative at the OraQuick® Support Center toll-free at 1-866-436-6527 or go to www.oraquick.com.

To find an HIV or PrEP Provider near you, use the QR codes below.

**HIV
Provider Locator:**

**PrEP
Provider Locator:**

**Still have more questions?
Find answers here: <https://oraquick.com/support>.**

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