



OraSure Technologies

Instant oral fluid drug testing



The only FDA-cleared 8-panel instant oral fluid drug test.

- Quick results in minutes
- Observed collection reduces adulteration
- Easy to administer
- Volume indicator to ensure complete specimen collection
- Allows for CDC-recommended 6 feet of separation
- Digital app available for results interpretation

OralTox[®]
MADE IN THE U.S.A.

Collect



Insert



Read



*Rapid results
Anytime, anywhere*

3 simple steps

Digital app results


Designed to be used with OralTox®, the OT-Scan app provides fast, accurate and secure results reporting by eliminating manual interpretation and effort.

Available for the detection of relevant drugs of abuse:

AMP, COC, MET, MTD, OPI, OXY, PCP, THC and more.



Test interpretation

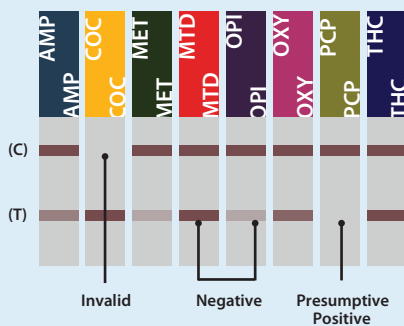

A **NEGATIVE result** is indicated by two colored bands appearing on the membrane. One band appears in the control region (C) and another band appears in the test region (T). The intensity of the test lines may vary. Any line appearing in the test region, regardless of intensity, color or size, indicates a negative result for that drug.

A **PRESUMPTIVE POSITIVE result** is indicated when a colored band appears in the control region (C) and no apparent colored band appears in the test region (T).



Negative results can be read as soon as the test and control lines appear on any test strip (often within 2 minutes).



Presumptive positive results are to be read at **10 minutes**. Interpret test as negative, presumptive positive or invalid. **Do not read test results after 20 minutes.**

Drug options and example panel configurations

- Amphetamine* (AMP)
- Barbiturates (BAR)
- Benzodiazepines (BZO)
- Cocaine* (COC)
- Methamphetamine* (MET)
- Methadone* (MTD)
- Opiates* (OPI)
- Oxycodone* (OXY)
- Phencyclidine* (PCP)
- Marijuana* (THC)
- **5 Panel:** OT-80502 (AMP, COC, MET, OPI, THC)
- **5 Panel:** OT-80508 (AMP, COC, MET, OPI, PCP)
- **6 Panel:** OT-80602 (AMP, COC, MET, OPI, PCP, THC)
- **7 Panel:** OT-80701 (AMP, COC, MET, OPI, OXY, PCP, THC)
- **8 Panel:** OT-80807 (AMP, COC, MET, MTD, OPI, OXY, PCP, THC)
- **10 Panel:** OT-81053 (AMP, BAR, BZO, COC, MET, MTD, OPI, OXY, PCP, THC)**



OraSure Technologies

www.OraSure.com 1.800.673.7873

*FDA 510(K) cleared **10 panel test not FDA-approved
Oratox® is a registered trademark of OraSure Technologies, Inc.
© 2020 OraSure Technologies, Inc., all rights reserved. INT0439 rev. 12/20