



ENSTRIKSYON SOU FASON POU PRAN TÈS LA

Fòk ou fè egzakteman sa yo diw fè a pou ou ka jwenn yon bon rezilta. Piga ou manje oubyen bwè anyen 15 minit anvan tès la, oubyen pa mete anyen pou ou lave bouch ou pou 30 minit anvan ou komanse fè tès la.

ATANSYON: Si ou sou tretman pou VIH (ARV) ou ka jwenn yon fo rezilta.



www.oraquickhivselftest.com

CREOLE

ORAQUICK[®]

HIV SELF-TEST

KIJAN POU OU SEVI AK TES LA




WAP BEZWEN YON BAGAY POU OU KONNEN KONBYEN TAN OU FE SOU TES LA



Sak la gen ladan: **kit tès la, kad pou kenbe tès la, ak enstriksyon sou fason pou pran tès la.**



Ti bwat tès la gen de sachè ladan li.

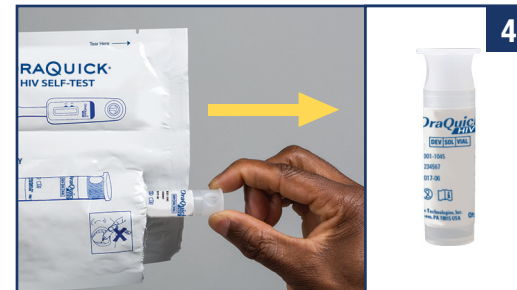

Ouvri sachè a ki genyen ti boutèy la ladan li.



Retire bouchon an.




PA vide likid la. **PA** bwè.



Mete ti boutèy la kanpe nan ti bwat la.




Chire sak ki gen aparèy tès la pou louvri li epi retire li. **PA** touche tanpon plat la ak dwèt ou. **PA** manje oswa bwè prezèvatif la.

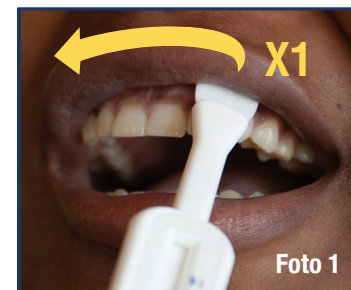

Byen peze epi pase pati ki plat la sou jansiv anlè bouch ou yon fwa – tankou ou wè nan premye foto a. Aprè sa fè menm bagay ak jansiv anba – tankou ou wè li nan dezyèm foto a.



Foure pati ki plat la nan boutèy la pou li rive anba boutèy la.

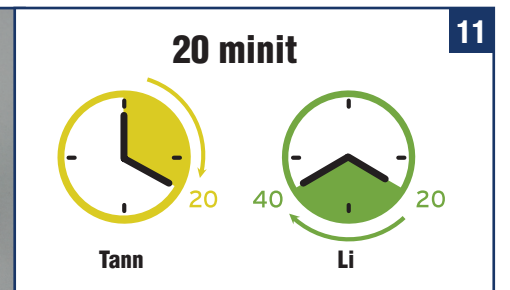


Kite li nan boutèy la pandan 20 MINIT anvan ou li rezilta yo. Fòk ou li rezilta yo apre 20 minit men pa plis ke 40 minit – rezilta yo pa bon apre 40 minit.

KOMAN POU W KONPRANN REZILTA YO



Li rezilta tès la yon kote ki byen klere

LE REZILTA YO POZITIF POU VIH

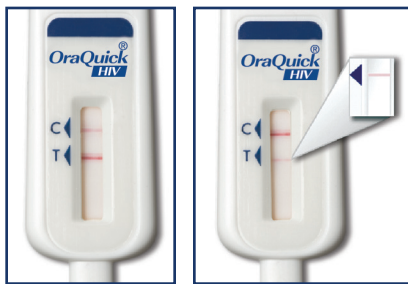

Si gen de (2) liy konplè, menm si koulè liy nan pal, sa vle di ou ka SEWOPOZITIF epi ou ka bezwen fè plis tès.



Pa pèdi tan . . .
Ale nan sant sante ki pre lakay ou a epi mande doktè oubyen mis la pou fè yon tès VIH pou ou.

LE TES LA PA BYEN MACHE



Lè ou pa wè yon liy bò kote "C" a (menm si ou wè yon liy bò kote "T" a), oubyen ou pa ka li tès la paske tout bagay wouj, ou ka konnen tès la pa byen mache epi fòk ou fè tès la ankò ak yon lòt aparèy. **Ou pa ka sèvi ak menm aparèy la ankò, ou bezwen jwenn yon lòt.**



Tès la pa byen mache.
Ale wè doktè oubyen mis nan sant sante ki pre prew la a epi mande pou yon tès VIH.

LE OU PA GENYEN VIH

SI OU LI REZILTA ANVAN 20 MINIT, GENYEN CHANS POU LI PA BA OU BON REZILTA



Si ou wè yon liy bò kote "C" a epi pa genyen liy bò kote "T" a, ou konnen ou pa gen VIH.

Fè yon tès VIH tanzantan. Si ou panse ou te nan yon sityasyon kote ou te ka pran VIH, ou met fè tès la ankò apre 3 mwa.

LE OU PA KWE NAN REZILTA TES LA

Ou pa konnen rezilta a oubyen ou pa kwè nan rezilta a.

Ale wè doktè oubyen mis nan sant sante ki pre prew la a epi mande pou yon tès VIH.

KIJAN POU OU JETE L

Retire tès la, mete bouchon sou ti boutèy la epi jete tout bagay yo nan fatra.

